


Thank you for purchasing the Living Earth Crafts/Earthlite Safe-Guard Treatment Shield. Please follow the instructions below to ensure a firm fit into your treatment table. Depending on the table you own, you may or may not require both fitting options.

If the dowels of the Safe-Guard Shield™ feel loose when sliding them inside the outlet receivers on your table, then follow steps below.

Step 1 : INSTALLING VELCRO STRIPS

1. Place the wooden base with the logo upside down.
2. Remove backing from velcro strip.
3. Wrap velcro around dowel and place as shown, at the base of the dowel.
4. Place one velcro strip on each dowel.

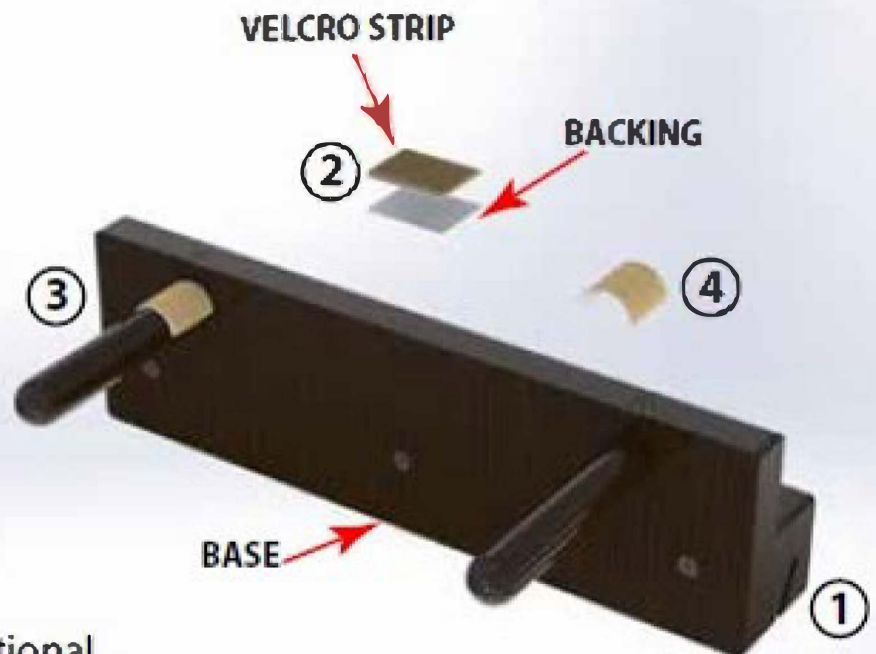

Note: we have included some additional velcro strips in case you need a replacement.

Step 2: INSTALLING SAFE-GUARD LOCKS

1. Push wooden receiver firmly against headrest outlet plate.
2. From under the table top, slide receiver lock firmly against top frame.
3. Tighten knob to lock in-place
4. Repeat for other lock.

